

多功能纸巾盒安装说明

Installation for multi-functional tissue box

产品用途/Product purpose




纸巾从此处放入, 纸从下面抽出
Place the tissue from here;
Tissues are drawn from below

可用于放纸巾/镜子/手机/杯子/车钥匙等小物件


Tissues/mirrors/cell phones/cups/car keys and other small objects can be stored here

零件清单/Components list

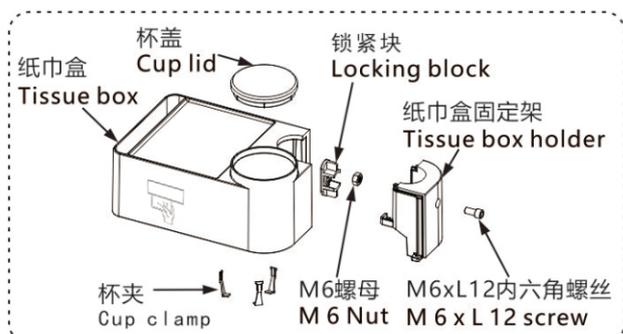

- | | |
|--------------------------------|------|
| 1、纸巾盒/Tissue box | 1PCS |
| 2、杯盖/Cup lid | 1PCS |
| 3、纸巾盒固定架/Tissue box holder | 1PCS |
| 4、锁紧块/Locking block | 1PCS |
| 5、杯夹/Cup clamp | 3PCS |
| 6、M6xL12内六角螺丝/M 6 x L 12 screw | 1PCS |
| 7、M6螺母/M 6 Nut | 1PCS |

产品使用安装步骤/Installation procedure

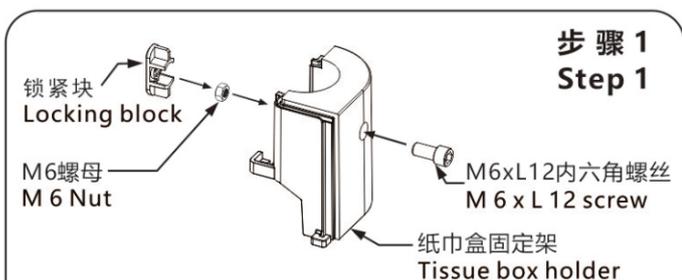

步骤 1 Step 1


锁紧块
Locking block

M6螺母
M 6 Nut

M6xL12内六角螺丝
M 6 x L 12 screw

纸巾盒固定架
Tissue box holder

将M6内六角螺丝插入纸巾盒固定架, M6螺母拧在螺丝上, 然后再放入锁紧块
Insert the M6 screw into tissue box holder, Fasten the M6 nut on M6 screw, and then insert the locking block

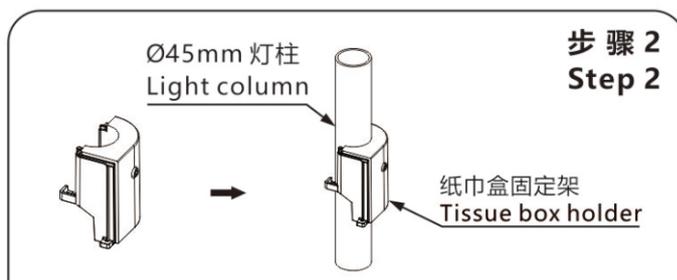


步骤 2 Step 2

Ø45mm 灯柱
Light column

纸巾盒固定架
Tissue box holder

将安装好螺丝的纸巾盒固定架扣在灯柱上
Fasten the screwed tissue box holder to the light column

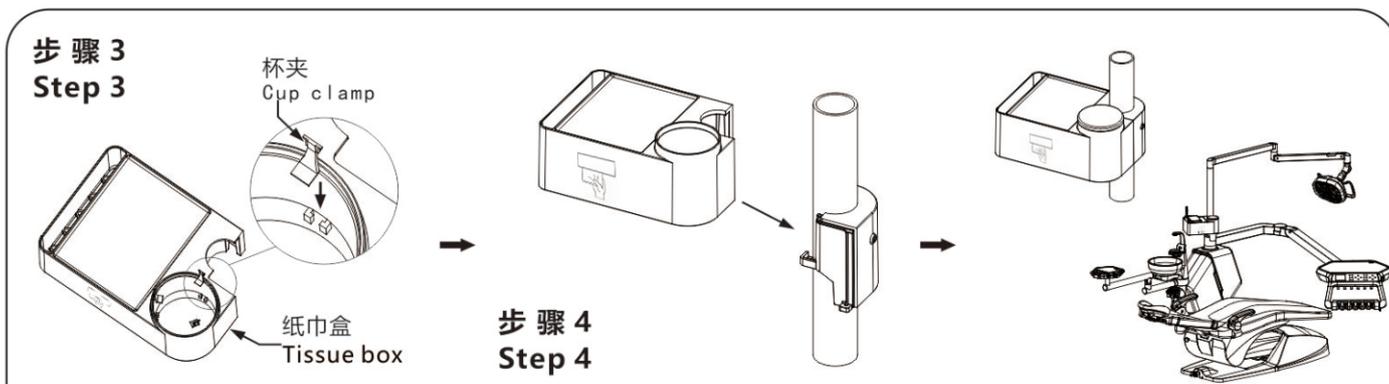

步骤 3 Step 3

杯夹
Cup clamp

纸巾盒
Tissue box

步骤 4 Step 4

将3PCS杯夹安装在杯桶内, 按下即可
Install 3PCS cup clamp in the cup barrel and press it down

从纸巾盒的底部往纸巾盒固定架的顶部往下插 (如右图)
Insert from the bottom of the tissue box down to the top of the holder (as shown on the right)

牙椅展示效果/安装完成效果
Installation completed